



GATEWAY PRODUCTS

SU-PER[®] flexWRAP

A Strong, Cohesive, Flexible Bandage!

An excellent pressure or support bandage.

SU-PER[®] flexWRAP is a self-adherent fabric bandage. It's elastic-like stretching quality allows for a firm and stable wrap that can be applied on the outside of wound dressings, leg wraps, or just about anything else you need to keep in place. It's an economical product to keep around the barn or trailer due to its many possible uses. See the many uses on page 2!

- Will not cut off circulation
- Easy to apply, but provides reliable holding power.
 - It adheres to itself, but not to hair and skin.
 - No fasteners, clips, or pins are needed.
 - Lightweight, porous, soft, and comfortable.

When using *SU-PER[®] flexWRAP* for securing bandages prior to competition, it is important to learn from someone who is experienced in using veterinary wraps. Pressure needs to be distributed around the fetlock and up the tendon properly for even pressure - uneven pressure may result in a bandage coming loose and causing serious problems while the horse is in motion. If the application of *SU-PER[®] flexWRAP* is too tight or too loose, simply unwind and work the wrap back onto the roll and try again!

Also do not apply *SU-PER[®] flexWRAP* over the top of an ointment or other medication unless the manufacturer recommends it, as this may lead to skin irritation or scurfing.

We also offer our *SU-PER[®] Wrap Cutter* for working with *SU-PER[®] flexWRAP*. The *SU-PER[®] Wrap Cutter* is designed to protect the leg while cutting off a flexible bandage, and is a great tool when removing *SU-PER[®] flexWRAP* from any part of the body safely and quickly.

LOOK FOR THE SU-PER BRAND!

P.O. Box 529 • Holly, Colorado, USA 81047 • info (888) 474-6367 • fax (888) 473-2954

SU-PER.com
SU-PERstore.com

A Strong, Cohesive Flexible Bandage!

- ◆ An excellent pressure or support bandage.
 - ◆ Will not cut off circulation if applied properly.
 - ◆ Easy to apply and provides reliable holding power.
- ◆ It adheres to itself, but not to hair or skin.
- ◆ No fasteners, clips, or pins are needed.
- ◆ Lightweight, porous, soft, and comfortable.

SUperior PERFORMANCE!

ROYAL BLUE



Find us on Facebook: [GatewayProducts](#)



Gateway Products
Holly, Colorado USA 81047



Find us on Instagram: [#GatewayProducts](#)



INSTRUCTIONS: Apply at approximately 50% tension. Do not apply too tightly as circulation may be impaired. Apply the last wrap with very little tension, and press firmly into place.

Wrap may be held in place with **SU-PER Vinyl Tape**.

Remove wrap with a **SU-PER Wrap Cutter** daily and reapply as needed.

CAUTIONS: Do not apply directly over open wounds. This product contains natural rubber latex which may cause allergic reactions.

FOR EQUINE USE ONLY



While people are constantly finding new ways to use FlexWRAP, here are some of the more common uses:

- Bandage wrapping for wounds.
- Supportive leg wraps or apply to your current leg wraps when the Velcro falls off.
- Protect hooves when being treated with a poultice or when a shoe is off.
- Provide support during racing or competition.
- Secure rundown patches or bandages.
- Wrap a pregnant mare's tail prior to foaling.
- Hold IV's in place on the horse's neck.
- Temporary tourniquet
- Fix old wooden rake handles to avoid splinters.
- Use it on tack to provide a stronger hold on buckles, snaps, and hooks as well as protect the horse from rubbing.
- Wrap your horses tail up to help it grow long, strong, and thick.
- Repair broken brushes and combs and other grooming tools.
- Stick over tops of nail heads or other sharp objects in stalls, trailers or pastures.
- Cover the fraying end of a lead rope or cover the chain on a lead shank.
- Cover the metal handle on water buckets during the winter to help keep your hands warm and prevent slippage.
- Keep your jeans folded at your ankle so your chaps will fit.
- To differentiate your tack from others.
- After grooming, to keep the legs clean until show.
- Tape around the cuffs of a rain jacket to keep the water from running in your arms.
- Protect against blisters for the rider, use on your hands or feet.
- Covering objects you don't want your horse to nibble on (posts, cords, etc.)